S. C. White
I.-V. Wagner
P. F. van der Stelt

CHAIRCOM INTERACTIVE
DECISION SUPPORT

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Decision Support for Interpretation of Radiographic Lesions and Clinical Management

Quintessenz Mediengruppe

DS X-RAY MANUAL

Table of Contents

1. Introduction
2. Installation instructions
3. General tips
4. Operating the program
 - 4.1. Anatomic location of anomaly, object density, vitality
 - 4.2. Viewing sample images
 - 4.3. Diagnostics
 - 4.4. Comparing magnified images
 - 4.5. Differential diagnostics
 - 4.6. Comparing different radiological changes
 - 4.7. Importing image files
 - 4.8. Index
 - 4.9. Closing the program

1. Introduction

DS X-RAY

Decision Support for Interpretation of Radiographic Lesions and Clinical Management

This process-integrated information system offers you visual documentation of radiological changes that can be used chairside.

There is often uncertainty in the diagnosis of radiologically visible changes (lesions). Relevant image material that is easy to manage should be available on the spot the moment a decision needs to be made. DS X-RAY lets you know how to differentiate harmless findings from those with a serious prognosis as early as possible.

The system includes more than 250 high-quality and optimized X-ray images accompanied by information on clinical features, diagnostics, differential diagnostics, and therapy of changes that can be shown in X-rays. Its contents are primarily geared to issues in dental practice. You can also import your own X-ray images into the program from digital X-ray equipments or data files, making direct comparisons possible.

2. Installation instructions

Before installation, please ensure that you have 40 MB of space available on your hard drive. The program is then installed as follows:

1. Insert the CD into the drive.
2. If the CD does not start automatically, open your CD drive through Windows Explorer and double-click the SETUP or SETUP.EXE application. The default path will be displayed. You can either accept the default path or select another directory where you wish to install DS X-RAY. If you select another directory, then you must enter the name of it.
3. There are two ways to start the program:
 - a: Double-click on the DS X-RAY icon on the Desktop, OR
 - b: Click Start – Programs – Decision Support – DS X-RAY.

3. General tips

If you wish to turn off the introduction, click the right button and select the option “Don’t show the intro again.” You can navigate the program’s various options by using the menu items that appear at the top of the screen:

File

“Import picture” (Ctrl + O) allows you to import an X-ray image from any directory on your computer. For more information about importing files, please see 4.7. With “Twain import” (Ctrl + T), you can transfer an image directly into DS X-RAY from the X-ray unit via the Twain interface. For this, the program must be connected to the device with the “Twain Select Source” command. By clicking “Print” (Ctrl + P), you can print out the diagnoses you have accessed as text. Select the printer you are using from the “Page setup” menu.

View

You can switch from the normal view (Ctrl + N) to a smaller window or full screen (Ctrl + F). You can also choose between 95% (Ctrl + 9), 85% (Ctrl + 8), and 75% (Ctrl + 7).

Settings

The “Sound On, Off” item in the Settings submenu allows you to specify whether your mouse clicks are audible or not. The Settings menu also allows you to choose your language for the user interface. If you wish to use a different language, simply click the one you want. You do not need to restart the computer after changing the language.

You can change the colors on the right side of the diagnostic screen by clicking “Set text color” or “Set text background color.”

Control

To restart the program, click “Restart” (Ctrl + R).

If you have already performed individual steps within the program, scroll to the option “Previous Step” (Ctrl + A) or “Next Step” (Ctrl + S) under Restart and select one. For added convenience, the arrows at the bottom of the screen may also be used for this purpose.

Extras

If you want direct access to an X-ray diagnosis, click “Index” (Ctrl + I). This will open a separate window displaying a list of diagnostic terms. For more information on diagnostic terms, please see 4.8. Select “Edit picture” (Ctrl + E) to edit an X-ray image after importing it. In addition to the standard color settings, you can enlarge or reduce the picture you’re working on with the magnifying glass icons and cut out parts of the image with the scissors or frame icon. You can also change the size and position of the frame. Simply drag the corner of the frame to the desired size and position and then activate the scissors icon. Click the disk -symbol to save the changes made to the imported picture. Return to the previous page and transfer the altered X-ray image by clicking the “Back” button.

Help

If you need help while using the program, you can access the DS X-RAY manual under “Help” (F1).

4. Operating the program

The following steps should be performed in the order listed:

4.1. Anatomic location of anomaly, object density, vitality

After starting the DS X-RAY program, the Start page will appear on your screen. Here you can choose between five options for locating the radiological change:

1. Tooth
2. Bone involving apex
3. Bone involving crown
4. Bone
5. Soft tissue

Depending on which location you've selected, you will be given one of three options for accessing sample X-rays.

- If you select the "Tooth" category, you will be offered three items in the next submenu to indicate which area you need to see.
- If you choose either the "Bone involving apex" or "Bone" category, you will be offered three options for describing the degree of density of the radiological change. After you have made your choice, you will be asked about the vitality of the tooth.
- The "Bone involving crown" and "Soft tissue" categories do not have any submenu items and take you directly to the relevant sample images.

Fig. 1 Start page

Fig. 3 Vitality

Fig. 2 Object density

4.2. Viewing sample images

Once you select the appropriate category, the program shows you various sample images. For side-by-side comparison, transfer your X-ray image created with a digital X-ray system into Decision Support by clicking either "Import picture" or "Twain import". The selected image is then automatically transferred to every page you access thereafter. You can change its size and position by clicking on it with your mouse. After saving the image, click on the X-ray example that most closely resembles your imported image..

4.3. Diagnostics

Once you've selected a sample image, other images will appear on the left side of the screen and your imported image will appear at the bottom of the screen. The right side of the screen will display the relevant diagnosis with a description of the characteristic features and recommended measures. The numbers in the text refer to the image samples shown on the left side from left to right and from the top down. You can enlarge (A+) or reduce (A-) the font by selecting one of the options beneath the text field. Moreover, you can print the displayed text by clicking "Print".

4.4. Comparing magnified images

Click on an image to enlarge it. Both your image and the sample image appear side by side on the screen, making a more detailed assessment possible. Click the image again to return to the Diagnostics screen.

Fig. 4 Sample images

Fig. 5 Diagnostics screen

Fig. 6 Magnification of images

4.5. Differential diagnostics

The smaller images on the lower left side of the screen represent differential diagnoses. Those highlighted in red contain particularly severe radiological changes. All differential diagnostic images can be accessed by clicking on sample images and the relevant Clinical Management. Clicking the arrows at the bottom of the screen takes you back to the previously accessed page.

4.6. Comparing different radiological changes

If a differential diagnosis is indicated for an oral mucosal change, you can display both clinical pictures parallel to each other. To do this, click on the "i" in the upper left of the screen next to the corresponding differential diagnosis image. In the next step, you will have the option of looking at the text or the image corresponding to the differential diagnosis. Return to the previous page by clicking the "Back" button.

Fig. 7 Differential diagnoses

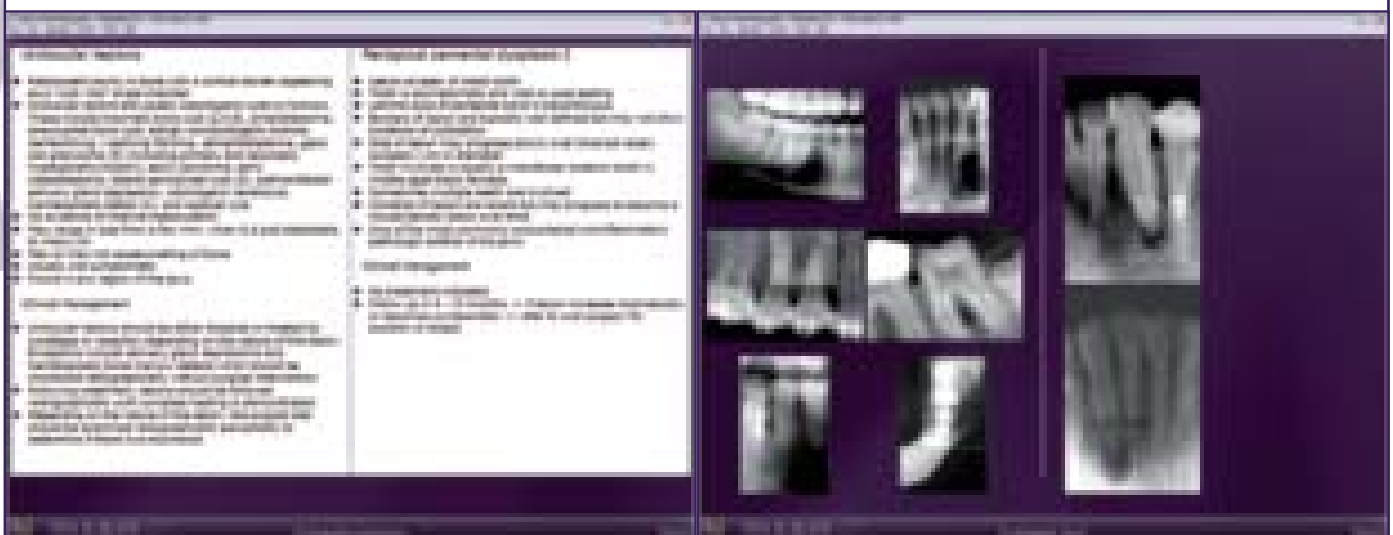
Fig. 8, Fig. 9 Comparing different radiological changes

4.7. Importing image files

The “Import picture” command (Ctrl + O) in the File menu allows you to import an X-ray image from any directory on your computer. After clicking on an image in the selected directory, that image is previewed in the right-hand field of the window. The dimensions of the image are displayed, and the image can be enlarged or reduced using the magnifying glass icons. Once you have chosen an image, click the “Open” button to import it. The last directory opened remains available for additional image imports. DS X-RAY supports the following image formats: BMP/JPG and TIF.

4.8. Index

Clicking “Index” (Ctrl + I) displays a new window with diagnostic terms listed in alphabetical order. Highlighted terms in blue can be activated and show you immediately the sample images and the corresponding description of the relevant radiological lesion (see 4.3.). If you run the mouse lightly over the terms, you will see a text field highlighted in yellow, which displays the description of the characteristic features and recommendations. The search function at the bottom of the window lets you find the desired clinical picture even more quickly and efficiently. After a letter is entered in the search field, the index is automatically positioned on terms starting with the selected letter.

4.9. Closing the program

Close DS X-RAY by clicking “Close program” (Ctrl + X) under the File menu or “Exit” at the bottom of the screen.

Fig. 10 Importing image files

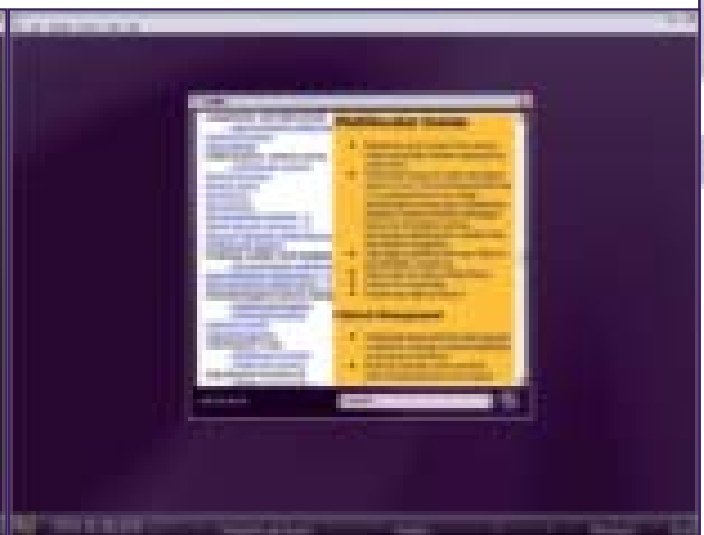
Fig. 11 Search for diagnoses

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